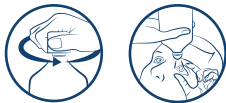
aluderm⁺

AQUA NIT[®] Augenspülung

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Optima Medical Swiss AG
Bundesstr. 7, CH-6300 Zug

Instructions for use - EN

First aid for rinsing the eyes, the conjunctival sac and the skin

Application

1. Twist off the cap
2. Rinse the affected area carefully
3. Keep the eye open with the fingers while rinsing, if necessary

Continue rinsing until all harmful substances are washed away completely. If required use additional bottles. If eyes are affected, remove contact lenses before application. Product is intended for first aid only. Consult a healthcare professional as soon as possible.

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EN

Indication and clinical benefit

First aid for rinsing the eyes, the conjunctival sac and the skin:

- after chemical burns caused by acids or lyes
- in case of irritation e.g. by pepper spray or tear gas
- in case of foreign bodies

Performance characteristics

Rinsing of the affected eyes and skin for removing irritating substances.

Shelf-life and storage

Do not freeze.

For single use only. Sterility is not ensured in case of re-used or damaged bottles.

Do not use after expiry date.

Contraindications and Precautions

Keep out of reach of children.

Do not use in the case of penetrating eye injuries (e.g. caused by a splitter).

Side-effects

Not known. In case of any serious incident that has occurred in relation to this medical device, report to the manufacturer or your competent authority.

Composition

Sodium chloride 9 mg/ml, Water for Injections

Notes

For adults and children. Children must be supervised by an adult.

Overdose is not possible as application is limited for first aid use.

Disposal

Disposal according to local requirements.

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Optima Pharmazeutische GmbH