M-EYE VISION

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M-EYE VISION

Eye spray with liposomes and pro-vitamin B5/dexpanthenol. Preservative-free.

Instructions for use:

Please read this information carefully because it will help you use this product properly.

Composition:

Lecithin 10mg/ml, pro-vitamin B5 (dexpanthenol), sodium chloride, ethanol, vitamin A - palmitate, purified water.

Indication and clinical benefit:

M-EYE VISION stabilises and protects the tear film and improves the moisturisation of the eye surface and the eyelids.

For treatment and alleviation of eye irritation by repairing damage to the tear film caused by external/environmental conditions.

To be used in case of dry, burning, itchy, tired, watery, irritated eyes or foreign body sensations.

M-EYE VISION makes wearing contact lenses more comfortable.

Performance characteristics:

Liposomes (lecithin) supplement the protective lipid layer of the tear film.

Liposomes and pro-vitamin B5 (dexpanthenol) moisturise the eye area and provide extra care as preventative protection for the skin.

Application:

Spray once or twice onto closed eyelids from about 10 cm away.

When using eye makeup, a slightly greater distance is recommended to prevent the makeup from running.

Generally, 3 to 4 times per day is recommended. For more severe discomfort, M-EYE VISION can be used more frequently without hesitation. Overdose is not possible due to the method of application and composition.

Can also be used while wearing hard or soft contact lenses.

Contraindications and precautions:

Should not be used if you are hypersensitive to any of the ingredients.

If eye drops/eye ointments are used, a time interval of about 10 minutes should be observed between the applications.

Do not spray onto open eyes.

If discomfort persists or worsens, seek medical advice. Keep out of reach of children.

Side-effects:

In very rare cases, temporary slight irritations (e.g. redness, swelling of the eyelids) may occur, which are limited to the area of application. They occur acutely and subside quickly and without consequences after cessation of use. If there is a severe adverse reaction in connection with this medical device, please report it to the manufacturer or the appropriate authority.

Notes:

For adults and children 2 years and older. Children should be supervised by an adult during use. For technical reasons, residual quantity may remain in the bottle. The bottle is overfilled to ensure the indicated amount is available for use.

Before initial use prime the pump by spraying several times to release the first dose.

Due to its liposomal formulation, M-EYE VISION is an opalescent liquid with a milky appearance and the characteristic natural scent of lecithin. In the course of storage, there may develop brownish turbidity/deposits, particularly in the area of the spring inside the pump.

This characteristic of the plant-based components (lecithin) does not influence the effectiveness or tolerability.

Shelf-life, storage and disposal:

Recommended storage: room temperature. Do not freeze. Can be used up to 6 months after first opening.

Do not use after expiry date.

Disposal according to local regulations.

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Optima Medical Swiss AG
Optima Sanità S.r.l.
Optima Pharmazeutische GmbH
Fountain House GmbH